



LED Tray Light & Teeth Whitening Kit

INSTRUCTIONS

STEP 1

Apply Teeth Whitening Gel

Insert the clear teeth retractor into your mouth to broaden your smile and then use the whitening pen (clear) to paint on a thin layer of whitening gel to the surface of your teeth. Make sure to remove any excess gel from your gums with a clean tissue or washcloth.

STEP 2

Activate Light

Plug one end of the adapter to your LED Mouth Tray and then plug the other end of the adapter into your desired power source to turn on. The LED lights from the mouth tray will illuminate to a neon blue when the tray has been successfully powered.

STEP 3

Whiten

Place the activated LED Mouth Tray into your mouth and lightly bite down on the tray ridge so that your lips close around the tray. Whiten for 30 - 60 minutes.

STEP 4

Deactivate Light

Unplug the LED Mouth Tray from your power source once you're finished whitening. Rinse your mouth with water, if desired.

STEP 5

Remineralize (Optional)

Once you've reached your desired whitening results (3-10 treatments), use the remineralizing gel to block out future stains.

STEP 6

Repeat

Repeat steps 1 through 5 every 4-6 weeks to maintain your Lucid White Smile.

CONTENTS:

- | | |
|---|---------------------------------|
| 1 - LED Mouth Tray | 1 - 2mL Teeth Whitening Gel Pen |
| 1 - Charging Adapter
(USB, iPhone & Android) | 1 - 2mL Remineralizing Gel Pen |
| 1 - Lucid Smile™ Beauty Bag | 1 - Cheek Retractor |
| | 1 - Instructions |

WARNINGS & TIPS:

Whitening Gel

- Keep out of the reach of children.
- Avoid contact with eyes and skin. If contact occurs, wipe the gel off and rinse the area with water.
- Do not use if you are pregnant or nursing, have poor dental health, wear braces, have had recent oral surgery, or are/may be allergic to peroxide.
- Use only under the supervision of a dentist.
- If you experience irritation/blistering on the lips/gums, discontinue use immediately, rinse the irritated area well and consult with your doctor/dentist (these effects will disappear within a few days by themselves).

LED Tray Light

- Do not submerge LED Mouth Tray in water while plugged into power source.
- Clean LED Mouth Tray gently, with wash cloth dampened with water.
- Do not use LED Mouth Tray for longer than 60 minutes at a time.
- Remove LED Mouth Tray from power source when you are done with use.
- Store LED Mouth Tray in clean, dry place.
- Keep out of the reach of children.

INGREDIENTS:

Whitening Gel Ingredients:

Propylene glycol, glycerin, hydrogen peroxide, urea peroxide, carbomer, triethanolamine, flavor, menthol, disodium EDTA

Remineralizing Gel Ingredients:

Active Ingredient: Sodium Fluoride

Other Ingredients: Glycerin, aqua, potassium nitrate, carbomer, sodium hydroxide, tricalcium phosphate, sodium saccharin, sodium benzoate, flavor, CI 42090

Manufactured for Lucid Smile™
by Beaming White, LLC.
1205 95th St, Ste A,
Vancouver, WA 98665

To reorder gels or any of our other teeth
whitening products, visit
www.beamingwhite.com/lucid-smile